Emmi® - steri 15

Bedienungsanleitung - Deutsch (3 - 7)
Operating Instruction - English (8 - 12)

Online-Shop:
www.ultraschall-welt.de

EMAG AG

Gerauer Str. 34, D-64546 Moerfelden-Walldorf

Tel.: +49 (0) 6105 - 40 67 00

www.emag-germany.de

Emmi® - steri 15

NEU: Heißluft-STERILISATOR

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Emmi® - steri 15

NEW: HOT air-STERILISER

- simple operation by rotary switch
- adjustable heating till + 200°C
- adjustable cleaning time 1-120 minutes or permanent operation
- excellent effect
- high efficiency
- incl. of aluminium insert
- EMAG-sterilisers are maintenance free

Product description

The hot air steriliser Emmi-steri 15 is made out of stainless steel and is resistant against heat and corrosion. It is completely isolated by fibreglass in order to avoid an overheating of the outside surfaces.

The sterilisation chamber is hermetically sealed by the cover due to a temperature resistant silicon sealing. Emmi-steri 15 has an insert and an insert basket made out of aluminium, both with a hole pattern in order to assure a good and sufficient air circulation. The equipment has a stainless steel encoated calorifier, which has a high efficiency factor and a low inertia.

1. temperature regulator
2. timer

Technical Data

Characteristics	Values
Height	210 mm
Depth	220 mm
Width	320 mm
Weight	4,3 Kg
Power supply	230 V~/50 Hz/1 A
Protection class	IP 32
Power	approx. 250W
Operating temperature range	+ 10 to + 30°C

Technical changes reserved

Scope of supply

- power supply cable
- removable aluminium insert
- removable aluminium insert basket
- two holders for removal of aluminium insert

Functional description

The operation of the equipment can be set to a certain time – max. two hours – or to permanent operation. The power switch is red and lights during operation. Additionally the equipment has a green control light which is alight during normal operation, and an orange control light which lights if heating is switched on and a red control light which will light in case temperature exceeds the max. limit and the safety circuit takes control over the temperature.

The wished temperature is adjusted with the temperature regulator. The equipment has an automatic temperature control which includes a self – adjusting capillary tube thermostat which has an auto-adjustment up to + 200°C. However, in case the temperature will nevertheless be exceeded a second safety control will be activated, a bi-metal switch, which then will take over the temperature control. If the bi-metal switch is activated the red alarm light in front of the equipment will go alight. In this case the service shall be contacted.

Conventional use

The equipment may only be used for the purpose as set forth in the catalogue and in the technical descriptions which means for the sterilisation of instruments and tools, and only in conjunction with external devices and components which are recommended or approved by EMAG AG. Any other use of the equipment is considered to be non conventional. The equipment may only be operated by persons who are familiar with the operation of the equipment. When the equipment is operated attention must be paid to persons in the surroundings of the equipment. In case of non conventional use EMAG AG will not be liable for possible damages and will not warrant for the correct and functionally work of the components. Part of the conventional use is also the observance of all information given in the operating manual. To this attention is drawn by the  sign on the type plate.

The correct and safe operation of the equipment implies appropriate transportation, storage, installation and assembly as well as careful handling and maintenance. The recurring inspections against BGV A3 are to be watched. The equipment must be installed in such a way that the power supply connector is always accessible.

Non conventional use

In case of non conventional use of the equipment danger for life and limb as well as danger for property may occur. Therefore a non conventional use is prohibited. In case of non conventional use EMAG AG will not be liable for possible damages and will not warrant for the correct and functionally work of the components. In order to avoid non conventional use of the ultrasonic cleaning equipment the following directions are to be considered:

- The equipment may only be operated with the voltage printed on the type plate on the backside of the housing (shockproof socket).
- The equipment must not be used for cleaning of contact lenses.
- Before any kind of handling of the equipment (i.e. transport, movement etc.) it must be assured that the power supply is disconnected and that the equipment chamber is at room temperature.
- No flammable or aggressive chemicals (i.e. acids etc.) must be used for cleaning.
- The equipment must only be operated in environments in which no explosive gases can occur.
- The equipment must only be operated indoor, in dry environment.
- In case of transportation damage, don't connect the equipment with main source.
- The equipment must be protected against humidity and wetness – no water must be filled into the interior, or must intrude into the equipment.

Operation

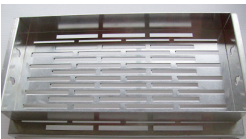
If the equipment – after unpacking – is transported it must be assured that that the equipment is carried in horizontal position and is hold below the bottom. Before connecting the equipment with the main source it must be checked that the voltage of the main source is the same than the one stated on the type label of the equipment (230V~ / 50 Hz). The equipment must only be installed on even areas in order to allow air to circulate below the equipment. Attention must also be paid that the back side of the equipment is at least 10 cm away from the wall or other obstacles in order to allow an adequate air circulation. The power supply cable is to be connected with the equipment and thereafter with the main source. Switch power supply switch to I (ON) and check whether the green control light is on.

⚠ *Note: If the equipment is switched on the first time a light smell of heated ceramic fibre may occur which may last for a short time. This is normal! Switch the time switch clockwise to the wished time, max. 120 minutes. Adjust the temperature regulator to the wished temperature and check whether the orange control light is on.*


⚠ *Note: The orange light may jitter a little bit until the adjusted temperature is reached. If the equipment is switched on at room temperature it will take approx. 30 minutes until the adjusted temperature is reached. The recommended operating temperature is +150°C to +180°C.*

Shall the equipment operate for more than 120 minutes the time regulator is to be adjusted anti clockwise to the position "∞". In order to get a correct sterilisation the insert must not be overloaded. The load must allow a good circulation of the hot air. When the adjusted temperature is achieved the sterilisation time shall be between 30 and 120 minutes, depending on legal requirements and / or specifications of the manufacturers of the items to be sterilised. The max. possible temperature should not be maintained for a too long time to avoid a damage of the items to be sterilised. Note: The light up of the red light (alarm) means that the temperature exceeds +220°C and that the safety thermostat has been activated. In this case the equipment shall be switched off and the service shall be informed. When the adjusted sterilisation time is elapsed the equipment stops automatically.

Insertion / removal of the inserts:

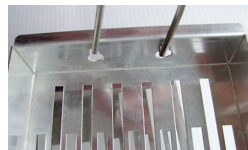Insert to be placed with opening down
Insertion basket to be placed on top with opening up

With the two holders the insert can easily be removed

Cleaning

It is recommended to clean the equipment only by using a soft cloth and pure water.
Never use acids or solvents! It is essential that the power is disconnected before cleaning!

Warranty

Warranty period for this product is 24 months – from date of purchase.

Defects occurring during the warranty period due to material or production failures will be corrected free of charge, either by repair or by replacement. In accordance with the applicable laws other compensation entitlements are excluded. Warranty will be void in cases of opened products, physical damage, misuse, modification, repair by unauthorised persons, carelessness and using the product for other purpose than its intended use. In any warranty case repair or replacement will not lead to an extension of the warranty period. In case of unjustified complaints, i.e. due to wrong installation or wrong operation we reserve the right to charge the costs occurring to us. For demands of warranty it is necessary to keep the purchase receipt showing the purchase date. In order to allow quick handling of warranty cases you should send the defect product directly to the below address, postage paid. By logistical reasons we cannot accept any shipment with postage not prepaid.

Thank you for your cooperation.

EMAG AG

Gerauer Str. 34, D-64546 Moerfelden-Walldorf

Tel.: +49 (0) 6105 - 40 67 00

www.emag-germany.de

date of purchase

stamp seller

changes reserved

© Copyright EMAG AG. All rights reserved. Manufactured in the European Community. The content of this Operating Instruction is subject to the copyright and it must not be copied neither in part nor in total. Changes, reductions, extensions and amendments of any publication or translation requires the prior written agreement of EMAG AG.

EMAG Technologies®

EMAG AG is certified to
DIN EN ISO 9001:2008

RoHS **CE**
(§6Abs.3, ElektroG)

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© Copyright EMAG AG. Alle Rechte vorbehalten. Hergestellt in der BRD/EU. Der Inhalt dieser Bedienungsanleitung unterliegt dem Urheberrecht und sie darf weder ganz noch teilweise kopiert werden. Veränderungen, Kürzungen, Erweiterungen und Ergänzungen jede Veröffentlichung oder Übersetzung durch Dritte bedarf der vorherigen schriftlichen Einwilligung durch EMAG AG.