



EndoSwab technical details

Product name: EndoSwab

Usage: Innovative, single use cleaning brush for flexible endoscope channel
Test for residual protein with Pyromol-E

Swab material: Highly absorbent polyester fibres

Sizes/Versions:

1.7: length 2.3m diameter 1.7mm
2.8: length 2.3m diameter 2.8mm
3.8: length 2.3m diameter 3.8mm
5.0: length 2.3m diameter 5.0mm

Endoswab	Best used for Endoscope channel
1.7 mm	1.5 – 1.7 mm
2.8 mm	2.0 – 2.5 mm
3.8 mm	2.6 – 3.0 mm
5.0 mm	3.1 – 4.5 mm

NOTE: Due to possible constriction or blockage of a channel, please always verify the right size of Endoswab with an initial test. Do not apply force when sliding the Endoswab through the channel.

Faulty material/operation cannot be ruled out 100%. After use always check if the tip of the swab is still attached and undamaged. EndoSwab is a single use product.

EndoSwabs are validated for the PEREG GmbH Pyromol-Test only, do not use in combination with unsuitable protein tests.

PEREG GmbH

30.04.2018

Martin Pfeifer
Managing Director